

Healthweigh® Indicators OIML Class III

Disabling Calibration Message

Product: Healthweigh Indicator OIML Class III

Issue: Unwanted calibration prompt

Solution: Set next verification date (MONT) to 0



Manuals are available from Rice Lake Weighing Systems at www.ricelake.com/manuals

Warranty information is available at www.ricelake.com/warranties

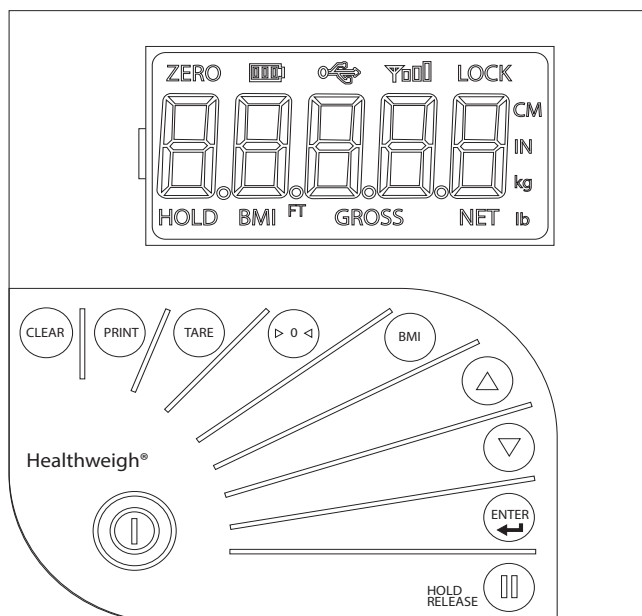


Figure 1. Healthweigh Indicator Display

A CNTL message may appear on an OIML Class III indicator to alert the user that it is time for a calibration. Use the following procedure to disable the CNTL calibration message:

1. Power on the indicator.
2. Immediately hold the **ENTER** button for three seconds until firmware ID number flashes.
3. Press the **BMI** button. Verification menu displays.
4. Press **ENTER** to access Verification menu.
5. Press **ENTER** to access MONT parameter.
6. Use the BMI and arrow buttons to set the MONT parameter to 0.
7. Press **Enter** two times to save.
8. Press **Enter** to select DONE. Scale will reboot after three seconds.



NOTE: The CNTL reminder is disabled after the procedure is complete. Menu may still continue to display 11 in the MONT parameter after successful procedure.



© Rice Lake Weighing Systems Content subject to change without notice.

230 W. Coleman St. • Rice Lake, WI 54868 • USA USA: 800-472-6703 • International: +1-715-234-9171