


Medical Scale Maintenance



Inspect the entire scale for any obvious signs of damage.

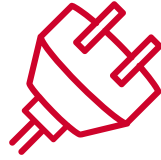

Check the AC adapter for cracked, frayed cords or broken, bent prongs.



Change batteries regularly to avoid leaks.

Cleaning Your Scale



Use mild soap and water to clean externally with a damp cloth.



Do not submerge the scale in liquid.



Avoid using Isopropyl alcohol to clean the display surface.

Checking Your Scale



Use NIST Certified test weights to check your scale for accuracy.



Check your scale using 10% of the scale's capacity.



Do not shock load or drop weights onto the scale.

RICE LAKE
WEIGHING SYSTEMS

HEADQUARTERS
230 West Coleman Street
Rice Lake, Wisconsin 54868 - USA
Tel: (715) 234 9171 | Fax: (715) 234 6967
www.ricelake.com