

# Wheelchair Scale Usage Guide



Model 350-10-3



Model 350-10-2

## Safety First

- Never leave a patient unattended
- Set the wheelchair brakes once the patient is on the scale
- Orient the scale for easiest entry and exit to ensure ramps are clearly accessible
- Only use ramps for entry and exit, do not enter or exit the scale over non-ramp sides

## Set Up for Accuracy

- The scale must be on a flat surface
- Ramps must not touch anything, including the floor, after loading
- Avoid factors that affect accuracy, such as fans, extra moisture or other issues
- Keep the scale clean of debris
- Tare wheelchair weight by placing the empty wheelchair on the scale and pushing the Tare button
- Gross weight should read 0.00 before the patient is weighed on the scale
- Remove accessories such as blankets, coats and handbags before weighing

## Operator Best Practices

- Never overload the scale past maximum capacity
- Make sure the scale has ample battery strength or use an AC adaptor
- Select LB/KG for desired unit of measure
- Avoid placing the scale on carpets or uneven surfaces

**RICE LAKE**<sup>®</sup>  
WEIGHING SYSTEMS

HEADQUARTERS  
230 West Coleman Street  
Rice Lake, Wisconsin 54868 - USA  
Tel: (715) 234 9171 | Fax: (715) 234 6967  
[www.ricelake.com/health](http://www.ricelake.com/health)